



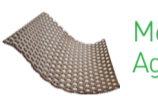
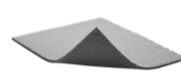

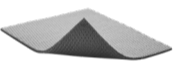

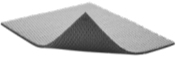








The Power of Gentle


Product selection guide for burns

Burns are painful and often life-changing for the burn victim. By reducing pain, distress and anxiety, we can support an effective healing process and improve outcomes. Our product portfolio does not make compromises. With a gentle Safetac® wound contact layer, our products provide effective care


and reduce additional trauma and suffering – every step of the way. This means undisturbed wound healing, improved cost-effectiveness, and a better patient experience¹⁻⁵.

Partial thickness burns			
Cleansing			
 Granudacyn®			
Exudate management / Antimicrobial barrier			
Exudate level	Normal-to-dress areas		Difficult-to-dress areas
Low	 Mepilex® Ag		 Mepilex® Ag
Medium	 Mepilex® Ag		 Mepitel® Ag + Secondary dressing  Mepilex® Transfer Ag + Secondary dressing
High	 Exufiber® Ag + Secondary dressing  Mepilex® Transfer Ag + Secondary dressing	 Exufiber® Ag + Secondary dressing  Mepilex® Transfer Ag + Secondary dressing	
Secondary dressings			
 Mextra® Very high exudate level		 Mesorb® High exudate level	
Fixation		Swelling management	
 Tubifast®  Tubifast® garments		 Tubigrip®	
Scar management			
 Mepiform®			

Donor site management


 Melgisorb® Plus + Secondary dressing

 or


 Exufiber® + Secondary dressing

Skin graft fixation

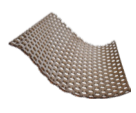
Low risk of infection:

 Mepitel® One + Secondary dressing

 or

 Avance® Solo

High risk of infection:

 Mepitel® Ag + Secondary dressing

Disclaimer: Always consult the Instructions For Use provided with the medical device before usage. Assortment and Indications for Use may vary depending on the country.

References:

1. Gee Kee EL, Kimble RM, Cuttle L, Khan A, Stockton KA. Randomized controlled trial of three burns dressings for partial thickness burns in children. Burns 2015, 41(5):946-955. 2. Silverstein P, Heimbach D, Meites H et al. An open, parallel, randomized, comparative, multicenter study to evaluate the cost-effectiveness, performance, tolerance, and safety of a silver containing soft silicone foam dressing (intervention) vs silver sulfadiazine cream. J Burn Care Res 2011, 32(6): 617-626. 3. Gee Kee EL, Stockton K, Kimble RM et al. Cost-effectiveness of silver dressings for paediatric partial thickness burns: An economic evaluation from a randomized controlled trial. Burns 2017, 43(4): 724-732. 4. Aggarwala S, Harish V, Roberts S et al. Treatment of partial thickness burns: a prospective, randomised controlled trial comparing Biobrane, Acticoat, Mepilex Ag and Aquacel Ag. J Burn Care Res 2020, 42(5): 934-43. 5. Tang H, Lv G, Fu J et al. An open, parallel, randomized, comparative, multicenter investigation evaluating the efficacy and tolerability of Mepilex Ag versus silver sulfadiazine in the treatment of deep partial-thickness burn injuries. J Trauma Acute Care Surg 2015, 78(5): 1000-1007.

Find out more at www.molnlycke.com

Mölnlycke Health Care AB, Box 13080, Gamlestadsvägen 3C, SE-402 52 Göteborg, Sweden. Phone +46 31 722 30 00. The Mölnlycke, Mepilex, Mextra, Tubifast, Exufiber, Mepitel, Mepiform, Mesorb, Granudacyn, Tubigrip, Melgisorb, Avance and Safetac trademarks, names and logotypes are registered globally to one or more of the Mölnlycke Health Care group of companies. © 2023 Mölnlycke Health Care AB. All rights reserved. HQIM004175

